ALLEVYN GENTLE BORDER

Silicone gel adhesive hydrocellular foam dressing

Description

ALLEVYN GENTLE BORDER Dressings have a triple-layered construction which combines an absorbent hydrocellular pad sandwiched between a perforated silicone gel adhesive wound contact layer and a highly permeable waterproof top film.

Mode of action

ALLEVYN GENTLE BORDER Dressings facilitates high fluid management and promotes a moist wound healing environment which is optimal for healing. It also helps to minimise the risk of maceration. The perforated silicone gel adhesive wound contact layer allows exudate to pass through to the foam layer. The silicone adhesive layer is gentle, suitable for use on fragile skin and makes it easy to apply, reposition and remove the dressing. The hydrocellular foam layer absorbs exudate away from the wound. The breathable top film allows evaporation and aids the prevention of fluid and bacterial contamination of the wound and is showerproof.

Indications

1. Wound management on shallow, granulating wounds, chronic and acute exudative wounds, full and partial thickness wounds including:

- Pressure Ulcers
- Leg Ulcers
- Diabetic Foot Ulcers
- Surgical wounds
- First and second degree burns
- Donor Sites
- Skin tears
- Fungating Ulcers

2. Pressure ulcer prevention on intact skin, including pressure ulcers caused by medical devices, as part of a pressure ulcer prevention protocol.

Precaution: Care should be taken when using the dressing under medical devices near the nose and mouth, where moisture can build up and affect adherence, leading to dressing movement.

For more information please visit: www.allevyn.com

Precautions

- Care should be taken when using the dressing under medical devices near the nose and mouth, where moisture can build up and affect adherence, leading to dressing movement.
- The use of ALLEVYN GENTLE BORDER Dressings, as part of a program of prophylaxis against the development of a pressure ulcer, does not preclude the need to continue to develop and follow a comprehensive pressure ulcer prevention protocol (e.g. regular repositioning, appropriate support surfaces, skin care and frequent skin assessments).
- Due to the gentle adhesive used, do not use this dressing to secure other medical devices to the patient.
- If reddening or sensitisation occurs discontinue use and consult a healthcare professional.
- Cutting the dressing will compromise the bacterial barrier and showerproof properties of the dressing.
- Do not immerse the dressing in water.
- Do not use with oxidizing agents such as hypochlorite solutions (e.g. EUSOL) or hydrogen peroxide as these can reduce the absorbency of the dressing.
- Single use only, and should not be re-used as cross contamination or infection may occur and the dressing may be ineffective.

- Once opened, do not retain unused dressings for application at a later date.
- If used on infected wounds, the infection should be managed as per local clinical protocol

ALLEVYN GENTLE BORDER Dressings are for healthcare professional use or for use by patients under the direction of a healthcare professional.

When required, healthcare professionals should provide appropriate advice to patients on how to apply, monitor the dressing, remove, and dispose of the dressing and when to contact a healthcare professional as detailed in this leaflet.

Instructions for use

1. Cleanse the application site. Ensure surrounding skin is clean, dry and free from excess hair.
2. Select appropriate dressing size. Ensure the absorbent pad covers the entire wound or area to be protected. The scale on the pouch is for guidance only. Avoid contact of the pouch with the wound.
3. Remove first liner and anchor the adhesive side of the dressing to the skin. If applying the sacrum dressing, place the narrow end a minimum of 2cm/ $\frac{3}{4}$ in. above the anal sphincter, then smooth the dressing over the sacrum.
4. Remove remaining liner/liners. Smooth over. If used on a wound, ensure absorbent pad covers the wound.
5. Smooth down the borders. Ensure there are no creases that could compromise the seal. The dressing can be repositioned as required.
6. ALLEVYN GENTLE BORDER Dressings can be cut especially to dress wounds on the heel, elbow and other awkward areas. Always use clean technique when cutting the dressing. Cutting the dressing will compromise the bacterial barrier and showerproof properties of the dressing. Ensure any exposed foam areas are covered with an appropriate film dressing taking care not to cover the entire dressing.

For further guidance on application of Heel and Sacrum see diagrams on dressing pouch or carton.

For necrotic or sloughy wounds, ALLEVYN GENTLE BORDER Dressings can be used in conjunction with INTRASITE GEL Hydrogel wound dressings.

Observation when used for pressure ulcer prevention: frequently inspect the skin for pressure damage by lifting back one corner, inspecting and then repositioning the dressing.

Removal: Lift one corner and slowly peel back until completely removed.

Dispose of ALLEVYN GENTLE BORDER Dressings according to local clinical protocols, or throw away hygienically after use.

Frequency of change

During the early stages of treatment inspect the ALLEVYN GENTLE BORDER Dressings frequently. ALLEVYN GENTLE BORDER Dressings can be left in place for up to 7 days, except the sacral area where dressings can be left in place for up to 5 days. ALLEVYN GENTLE BORDER Dressings should be changed depending on the condition of the wound and surrounding skin, or until exudate is visible as strike through and approaches to within 0.5cm/3/16in. of the edge of the dressing pad, whichever is sooner. The decision of when to change should be dependent upon clinical assessment and local protocols under the guidance of a healthcare professional. Complaints@Smith-Nephew.com

If during the use of this device you believe that a serious incident has occurred, please report it to the manufacturer and your competent authority. Complaints@Smith-Nephew.com

Product availability

Code	Size	Availability
66800269	7.5cm x 7.5cm / 3in. x 3in.	Carton of 10
66800270	10cm x 10cm / 4in. x 4in.	Carton of 10
66800900	10cm x 20cm / 4in. x 8in.	Carton of 10
66800264	10cm x 25cm / 4in. x 9.8in.	Carton of 10
66800265	10cm x 30cm / 4in. x 11.8in.	Carton of 10
66800272	12.5cm x 12.5cm / 5in. x 5in.	Carton of 10
66802142	12.5cm x 12.5cm / 5in. x 5in.	Carton of 3
66800975	15cm x 15cm / 6in. x 6in.	Carton of 10
66800273	17.5cm x 17.5cm / 7in. x 7in.	Carton of 10
66800897	Sacrum Dressing 16.8cm x 17.1cm / 65/8in. x 63/4in.	Carton of 6
66800898	Sacrum Dressing 16.8cm x 17.1cm / 65/8in. x 63/4in.	Carton of 10
66801031	Sacrum Dressing 21.6cm x 23cm / 81/2in. x 9in.	Carton of 10
66800506	Heel 23cm x 23.2cm / 9in. x 91/8in.	Carton of 5

Trade Marks of Smith+Nephew. Date of issue 04/2024. Revision 02.

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°Trade Marks of Smith+Nephew. ©Smith+Nephew
Smith & Nephew Medical Limited,
Smith & Nephew Operations B.V.