
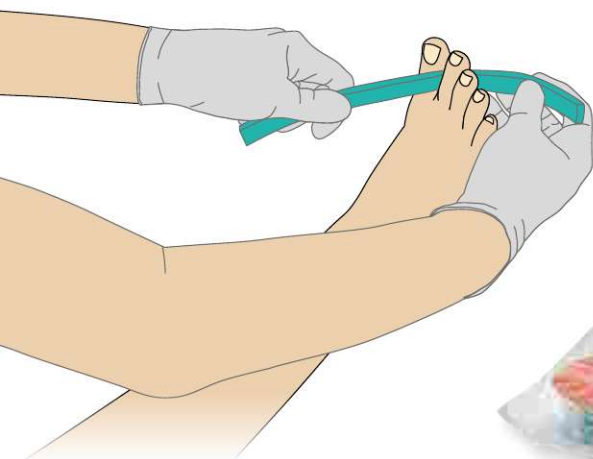


LIGASANO® Wundputzer®

Wound Cleaning Sponge

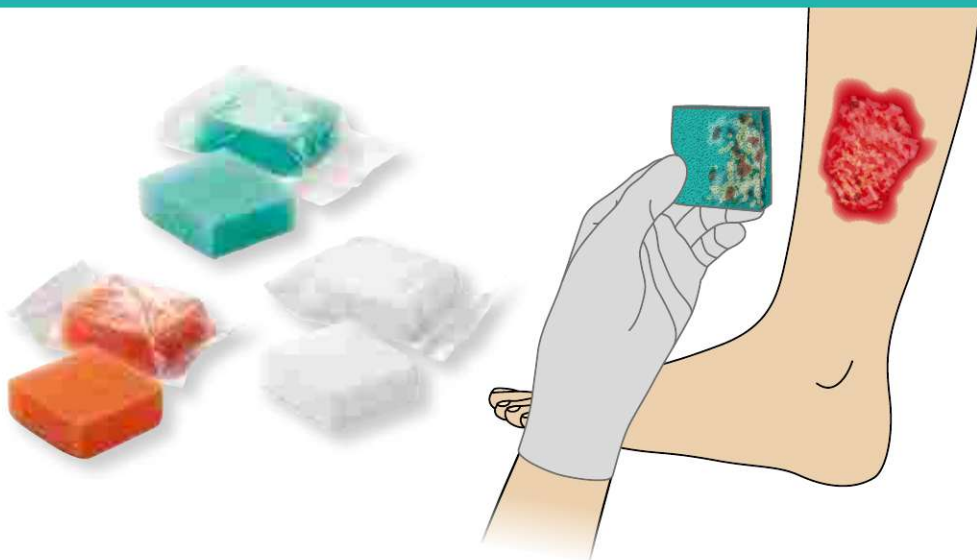

LIGASANO®
Simply Works

LIGASANO® Wundputzer®

LIGASANO® sterile with the additional name Wundputzer® (wound cleaning sponge) usually remains in the wound only for a short time to clean it.

There are available three types: LIGASANO® green (=intensive), LIGASANO® orange (=medium) and LIGASANO® white (=soft).

A clean wound is absolutely required for wound healing and wound documentation!



LIGASANO® green sterile Wundputzer® wound cleaning sponge intensive

Removes very persistent, viscous, firm and/or necrotic coatings, may absorb a lot of detritus.



LIGASANO® orange sterile Wundputzer® wound cleaning sponge medium

Removes viscous, firm and/or necrotic coatings, may absorb plenty of detritus.



LIGASANO® white sterile Wundputzer® wound cleaning sponge soft


Removes soft coatings and biofilm.

- ▶ **Easy in application:** the size 5 x 5 x 2 cm fits comfortably in the hand.
- ▶ **Effective & economic:** you can vary the intensity of the wound cleaning effect with the pressure power - adjusted to the need of your patient. Because of its structure, the LIGASANO® Wundputzer® does not disperse the coatings in the wound but retrieve it.
- ▶ **Hygienic:** either three pieces in one peel-pack (1x green, 1x orange, 1x white) or single packed. Thus you can clean the wound from the coated surface to the bottom always with a fresh LIGASANO® Wundputzer® wound cleaning sponge. The LIGASANO® wound cleaning sponges are useable multilateral.

Please note: a mechanical debridement may cause pain, even with the LIGASANO® Wundputzer® wound cleaning sponge! Please pay attention to sufficient analgesia, if necessary. A moistening with wound irrigation solution can make sense. In the case of patients during a therapy with anticoagulants (e.g. Marcumar® or acetylsalicylic acid) an appropriate risk assessment has to be made. The wound treatment with LIGASANO® has to be made by physicians or medically trained personnel.

Please read the instruction manual before application! You find it in every original packaging.

... and wound healing may begin

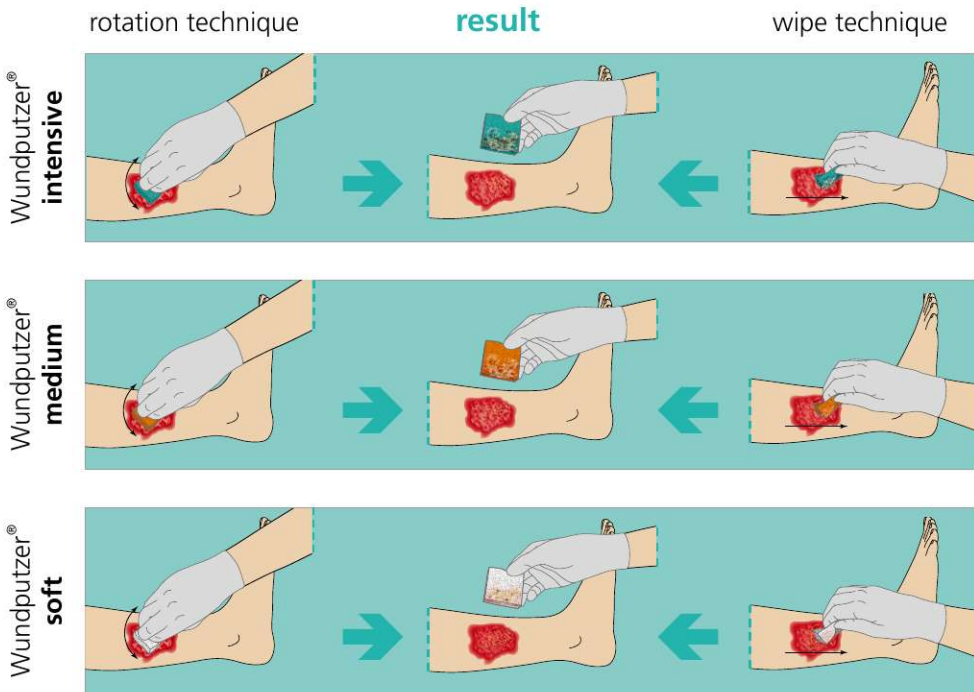

LIGASANO® Wundputzer® wound cleaning sponge interdigital

Where cleaning is difficult e.g. between toes, in wound pockets or in undermined wounds, the LIGASANO® Wundputzer® wound cleaning sponges interdigital will help you! They have the same properties as our regular wound cleaning sponges and are only adapted in form for special applications.



LIGASANO® Wundputzer®

Wound Cleaning Sponge

Available Packages:



Item no.	Shipping Unit
16814-030	30 pcs. 5 x 5 x 2 cm green sterile ●
36814-030	30 pcs. 5 x 5 x 2 cm orange sterile ●
15342-030	30 pcs. 5 x 5 x 2 cm white sterile ○
16815-005	5 peel-pack Wundputzer® wound cleaning sponges each with - 2 pcs. LIGASANO® green 5 x 5 x 2 cm sterile ● - 1 piece LIGASANO® white 5 x 5 x 2 cm sterile ○
36815-005	5 peel-pack Wundputzer® wound cleaning sponges each with - 1 piece LIGASANO® green 5 x 5 x 2 cm sterile ● - 1 piece LIGASANO® orange 5 x 5 x 2 cm sterile ● - 1 piece LIGASANO® white 5 x 5 x 2 cm sterile ○



Item no.	Shipping unit
16812-020	20 pcs. 30 x 1.5 x 0.6 cm green sterile ●
36812-020	20 pcs. 30 x 1.5 x 0.6 cm orange sterile ●
15372-020	20 pcs. 50 x 1.5 x 0.4 cm white sterile ○

- = LIGASANO® white
- = LIGASANO® orange
- = LIGASANO® green

LIGASANO®
simply works