
ADVANCED WOUND CARE

<p>EC REP</p> <p>Ahera Ltd., Tower Business Centre, 2nd Fl., Tower Street, Swatara, BKR 4013 Mella ☎ +44 (1) 926 800 153 DOC. No: BDH-DRAWTEX-UF-09 (REV 12)</p>	<p>Drawtex® and LevaFiber™ are trademarks of Beier Drawtex Healthcare (PTY) Ltd. Patented and other patents pending. Product names are used for reference purposes only and are trademarks of their respective owners.</p>
<p>BEIER DRAWTEX HEALTHCARE (PTY) LTD. 40 Gillits Road, Pinetown, 3610, South Africa ☎ +27 (0) 31 710 0400 ☎ +27 (0) 31 700 3126 ✉ info@drawtex.com www.drawtex.com</p>	

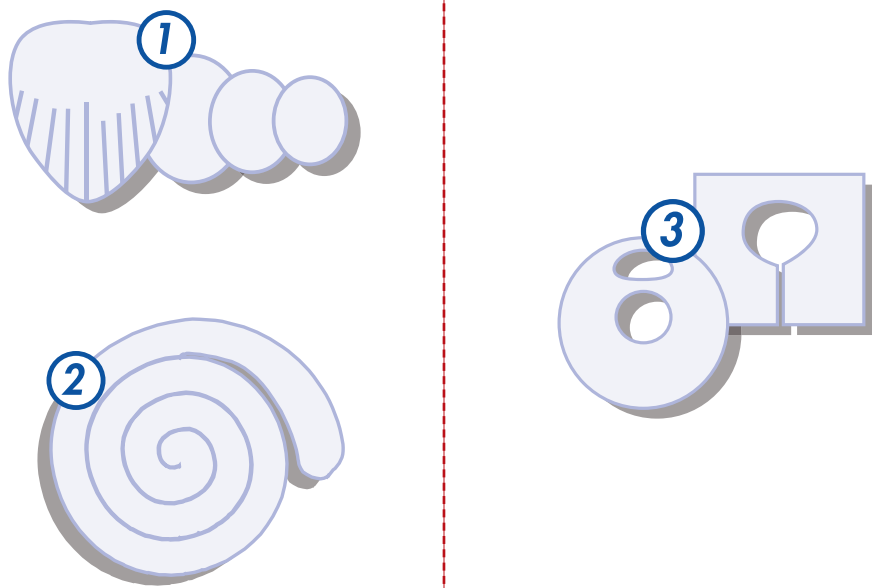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en DRAWTEX® HYDROCONDUCTIVE WOUND DRESSING

COMPOSITION
Drawtex Hydroconductive Wound Dressings with LevaFiber Technology are made up of natural and synthetic fibre: Viscose, Cotton and Polyester. Drawtex Wound Dressings are non-medicated.

MECHANISMS OF ACTION
Drawtex lifts, holds and transfers exudate, wound debris, bacteria and harmful MMPs away from the wound bed through three mechanisms of action: capillary, hydroconductive and electrostatic. Drawtex supports the body's healing process and aids effective wound bed preparation without damaging newly formed tissue.

INTENDED PURPOSE
Drawtex is a hydroconductive wound dressing intended for the management of medium to high levels of exudate in acute and chronic wounds.

CONTRA-INDICATIONS
DO NOT USE IF ARTERIAL BLEEDING IS PRESENT!

PRECAUTIONS
• A wound contact layer should be used on burns, as well as low-exuding wounds, to prevent adherence to the wound bed (e.g. paraffin gauze or silicone wound contact layer).

Cut: Drawtex may be cut to conform to wound shape. Any side of Drawtex can be used against the wound bed.

Apply: For low exuding or dry wounds, apply a nonadherent (perforated) dressing before applying Drawtex. For best results, ensure dressing has direct contact with the wound bed.

Layer: For moderate to highly exuding wounds, apply Drawtex directly to wound bed. For highly exuding wounds, apply additional layers as necessary.

Cover: Cover with a secondary dressing or bandage of choice.

Change: Change Drawtex every 1 to 3 days, as necessary. Once exudate is under control, dressing may be changed less frequently (not to exceed 7 days).

applying the dressing, cleanse the wound area with an appropriate wound cleanser. Fit the Drawtex Tracheostomy Dressing around the tracheostomy tube; it does not matter which side is in contact with the skin. Interlock the dressing above the tracheostomy tube. If required, the dressing may be cut to fit the tracheostomy shape. Once the dressing is saturated it should be changed immediately.

To Remove the Drawtex Tracheostomy Dressing:
Remove the retention dressing or bandage carefully and lift the dressing away from the wound.

CUTTING INSTRUCTIONS
1. Sacral Shape:
The heart shape is excellent for dressing wounds on the sacrum as well as on other anatomically shaped wounds such as the hip. In order to enhance conformability in these types of wounds, one can cut vertically into the dressing. The dressing can be cut into an oval or round shape, to fill a cavity. Multiple layers of Drawtex can be applied in the cavity wound until the dressing is at least 2 layers above the wound edges.

2. Spiral Shape:
The spiral shape can be used for many different types of wounds - cavity wounds, amputations and abdominal wounds. Depending on the levels of exudate,

SAFETY
• Drawtex is for external use only.
• Drawtex is intended for single use only. Do not reuse.
• Drawtex is sterile unless the packaging is opened or damaged.
• Drawtex should be stored in a cool, dry place.
• Do not use Drawtex if the package is open or damaged.
• Any serious incident occurring in relation to use of the device should be reported to the manufacturer or relevant competent authority.

DATE OF PUBLICATION OF THE INSTRUCTIONS FOR USE:

MANUFACTURED AND PACKAGED BY:
BEIER DRAWTEX HEALTHCARE (PTY) LTD

Head Office: 40 Gillits Road, Pinetown, 3610, South Africa.
T: +27 (0) 31 710 0400
F: +27 (0) 31 700 3126
E: info@drawtex.com
W: www.drawtex.com

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• After removal, the existing dressing should be disposed of according to local regulations.
• Reuse could lead to cross-contamination, inadequate performance, infection or inflammatory response.

SKIN COMPATIBILITY
Drawtex has been tested against DIN EN ISO 10993 and meets the requirements for cytotoxicity, skin irritation and sensitization.

INSTRUCTIONS FOR USE
Before applying the dressing, cleanse the wound area with an appropriate wound cleanser.

If Drawtex is adherent, irrigate with saline for easy removal.

Note: Good wound care principles require caregivers to use aseptic technique, as well as wearing sterile gloves, while completing dressing changes. Intended for use by healthcare professionals.

Using Drawtex Tracheostomy Dressings:
The product is for single use only and is supplied sterile. If the immediate product packaging is damaged, do not use. Before

extra layers of Drawtex may be used in conjunction with the spiral shape.

3. Stoma Shape:
This shape is most suited for areas where there are high levels of exudate leaking onto healthy skin.

INDICATED FOR:
Drawtex is indicated for the management of a wide range of acute and chronic wounds such as skin tears, traumatic and / or penetrating injuries, superficial, partial thickness burns (first and second degree), complex, surgical, and dehisced post operative wounds, pressure ulcers, venous leg ulcers, diabetic foot ulcers, cavity wounds, non-healing or maintenance wounds.

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