**ALLEVYN LIFE****Tootja kontaktandmed / Manufacturer contact details**

Smith & Nephew Medical Limited  
101 Hessle Road, Hull HU3 2BN, England

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[complaints@smith-nephew.com](mailto:complaints@smith-nephew.com)  
[www.smith-nephew.com](http://www.smith-nephew.com)  
[www.smith-nephew.com/patents](http://www.smith-nephew.com/patents)

Date of issue: 11/2025  
Revision 03  
Printed in: China  
Document ref: 10601957 / 48818

## ENGLISH (EN)

### ALLEVYN LIFE - Silicone gel adhesive composite hydrocellular foam dressing

#### Description

ALLEVYN LIFE Dressings are silicone gel adhesive composite hydrocellular foam dressings. The absorbent pad has a foam layer and lock away core made from super absorbent material. There is a discretion layer over the centre of the pad, the dressing is breathable and it has a waterproof top film.

#### Mode of Action

ALLEVYN LIFE Dressings have a perforated silicone adhesive wound contact layer which enables exudate to pass through to the foam layer. The silicone adhesive layer is gentle, suitable for use on fragile skin and makes it easy to apply, reposition and remove.

The foam layer absorbs exudate away from the wound and peri-wound and the EXULOCK hyper-absorbent layer absorbs exudate and helps it to spread laterally across the dressing, gradually utilising the entire dressing area to lock in exudate, helping to control odour.

The breathable top film allows evaporation and helps prevent bacterial contamination of the wound. Together the absorbent and breathable layers promote a moist wound environment which is optimal for healing of low to highly exuding wounds.

The EXUMASK Masking Layer visually masks unsightly exudate and provides a visual indicator of when the dressing needs to be changed.

#### Intended Purpose

1. To provide a moist wound environment which is optimal for healing.
2. To prevent pressure injury on intact skin as part of a pressure injury prevention protocol.

#### Indications

1. Wound management on shallow, granulating wounds, chronic and acute exudative wounds, full and partial thickness wounds including:

- Pressure ulcers
- Leg ulcers
- Diabetic Foot Ulcers
- Surgical wounds
- Superficial and partial thickness burns
- Skin graft donor sites
- Skin tears
- Fungating Ulcers

2. Pressure injury prevention on intact skin as part of a pressure injury prevention protocol.

#### Precautions

- Do not cut the absorbent pad as internal dressing components may contaminate the wound leading to delayed healing.
- Do not immerse the dressing in water.
- Do not use with oxidising agents such as hypochlorite solutions (e.g. EUSOL) or hydrogen peroxide as these can reduce the absorbency of the dressing.
- Should any signs of maceration, discomfort, sensitisation or irritation appear during use, or if the wound worsens or healing stalls, consider discontinuing use. If symptoms do not improve consult a healthcare professional.

- The use of ALLEVYN LIFE Dressings as part of a prophylactic pressure injury prevention therapy, does not preclude the need to continue to follow a comprehensive pressure injury prevention protocol (e.g. regular turning, appropriate mattress and skin care).
- ALLEVYN LIFE Dressings are single-use only and should not be re-used as cross-contamination or infection may occur and the dressing may be ineffective.
- Once opened, do not retain unused dressings for application at a later date.
- If used on infected wounds or wound becomes infected, the infection should be inspected and treated as per local clinical protocol.

If during the use of this device you believe that a serious incident has occurred, please report it to the manufacturer and your competent authority.

Dressings are for healthcare professional use or for use by patients under the direction of a healthcare professional.

When required, healthcare professionals should provide appropriate advice to patients on how to apply, monitor the dressing, remove, and dispose of the dressing and when to contact a healthcare professional as detailed in this leaflet.

For more information please visit: [www.smith-nephew.com](http://www.smith-nephew.com)

### **Instructions for use**

Cleanse the application site. Ensure surrounding skin is clean, dry and free from excess hair.

1. Select appropriate dressing size. Ensure the absorbent pad covers the entire wound or area to be protected. The scale on the pouch is for guidance only.
2. Remove first liner and anchor the adhesive side of the dressing to the skin.
3. Remove remaining liner / liners. Smooth over. If on a wound ensure absorbent pad covers the wound. Do not stretch the dressing.
4. Smooth down the borders. Ensure there are no creases that could compromise the seal. The dressing can be repositioned as required.

### **Additional use and change guidance**

ALLEVYN LIFE Dressings can be used in conjunction with INTRASITE GEL Hydrogel for necrotic or sloughy wounds.

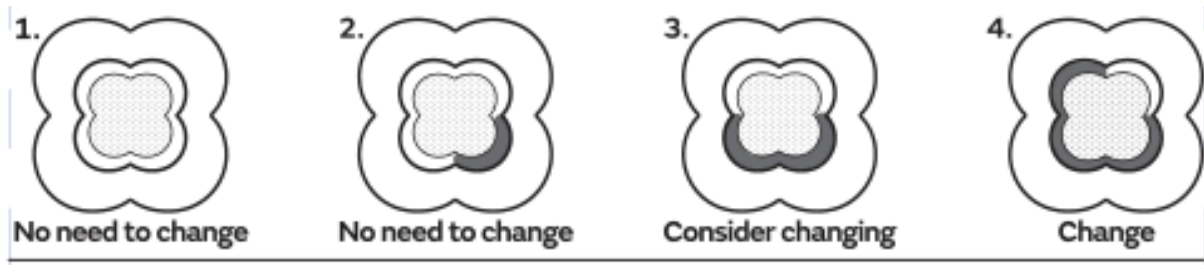
Observation: When used for pressure injury prevention, frequently inspect the skin for pressure damage by lifting back one corner, inspecting and then repositioning the dressing.

Removal: Lift one corner and slowly peel back until completely removed to avoid skin injury.

Improper disposal of dressing and packaging can cause slips, contamination, and infection; dispose of them per local clinical protocols or hygienically after use.

Frequency of change: During the early stages of treatment inspect the dressing frequently. Dressings can be left in place for up to 7 days. Dressings should be changed depending on the condition of the wound and surrounding skin or when exudate covers 75% of the change indicator. Consider changing if the exudate covers more than 50% of the change indicator. The diagram below is for guidance only and the decision of when to change should be dependent upon clinical assessment and local protocols under the guidance of a healthcare professional.

### **Change indicator guide**

Trade Marks of Smith+Nephew. Date of issue 11/2025. Revision 03. This product may be covered by one or more US Patents. [www.smith-nephew.com/patents](http://www.smith-nephew.com/patents)

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